

User Manual

MagniLink S PREMIUM 2 - PC

LVI America Inc.

302 Saunders Road, Suite 200 Phone: (888) 781-7811
Riverwoods, IL 60015, USA

info@lviamerica.com
www.lviamerica.com

LVI Low Vision International

Verkstadsgatan 5 Tel: +46 470-727700
SE-352 46 Växjö Faks: +46 470-727725

info@lvi.se
www.lvi.se

Table of Contents

1	About LVI.....	5
2	Introduction to the MagniLink S.....	6
3	Information.....	8
3.1	Reuse, installation and maintenance.....	8
3.2	Cleaning	8
3.3	Safety information.....	8
4	Unpacking	10
4.1	Packaging.....	11
5	Accessories	12
6	Hardware	15
6.1	Installation of hardware	15
6.2	Installation of docking station.....	16
6.3	Using the hardware.....	17
7	MagniLink PCViewer.....	20
7.1	Minimum system requirements	20
7.2	Recommended system requirements	20
7.3	Installation of software	20
7.4	Licensing.....	23
7.5	Using the software	26
7.6	Update software	46
8	Working with the reading camera.....	49
9	Troubleshooting	50
10	Technical information.....	51
11	Declaration of Conformity	52

1 About LVI

LVI Low Vision International AB was founded in 1978 and is one of the world's leading manufacturers of magnifying TV- and computer systems for visually impaired. New products are developed in close cooperation with users and professional personnel of vision rehabilitation. LVI is a Quality System Certified ISO 9001 and Environmental System Certified ISO 14001 company.

The information in this manual has been carefully checked and is believed to be accurate. However, LVI assumes no responsibility for any inaccuracies that may be contained in this manual. In no event will LVI be liable for direct, indirect, special, exemplary, incidental, or consequential damages resulting from any defect or omission in this manual. In the interest of continued product development, LVI reserves the right to make improvements in this manual and the products it describes at any time, without notice or obligation.

2 Introduction to the MagniLink S

Thank you for choosing this video magnifier. We aim to ensure that you are completely satisfied with your choice of supplier and reading aid.

The MagniLink S is a portable camera system with autofocus, which can be connected to a computer or directly to an external monitor. The system is controlled via a keyboard and/or control box. MagniLink S is available in a number of different models. Choose model in the following steps:



Choose reading camera HD or FHD.

Model	Camera
MLS-HD-2	HD (720p)
MLS-FHD-2	FHD (1080p)

Choose among the following accessories.

Product code	Accessory
A	Text To Speech for PC
B	Text To Speech for Mac

	The systems complies with all CE certification requirements for medical devices in accordance with EU regulation 2017/745.
---	--

3 Information

3.1 Reuse, installation and maintenance

The device may only be technically overhauled and prepared by an authorized company. Re-use may only be carried out after appropriate inspection and cleaning / disinfection.

3.2 Cleaning

Before equipment cleaning and maintenance note:

- Disconnect the device from the power supply before cleaning.
- Use a slightly damp (not wet) cloth and a cleaning / disinfectant agent intended for monitors and computers.
- Do not allow any liquids to enter the device.

3.3 Safety information

The reading aid is only intended for indoor use and must be used in accordance with the following safety instructions.

- Use only the power adaptor supplied with MagniLink S. If the wrong type of adaptor is used for operation of MagniLink S, this can cause damage to the system.
- Units such as HDMI monitors and/or computers that are to be connected to the system must comply with the appropriate safety regulations for their country of use.
- Never turn on or use the product if it has been exposed to water, rain or other liquids. Pull out the USB cable and allow trained service personnel to carry out an inspection of the product before it is used again.
- Place the power cord in such a way that you can neither trip over it or that crush damages can occur.
- If the product is moved from the cold into a warm room, condensation (moisture) can form inside the unit or on the camera lens. To prevent problems with condensation, the apparatus should be installed in the room at least 30 minutes prior to usage.
- If the system requires cleaning, pull out all cables and use a slightly damp (not wet) cloth. Use only water or a cleaning agent that is intended for monitors and computers.

- Do not rely 100 per cent on the results of an OCR conversion; the OCR engine can misinterpret the text. This is particularly important to bear in mind when processing sensitive information such as a medical prescription.
- Make no further adjustments, even if the product does not function normally after following the functional descriptions. Pull out the USB cable and contact service personnel or the product distributor. Only use spare parts that are specified by LVI

If in doubt, contact your supplier.

Warning



MagniLink S is supplied packed in a plastic bag. To avoid risk of suffocation, keep the bag out of the reach of children and infants.

4 Unpacking

The system is supplied in a box with the following contents:



- Reading camera
- Removable USB/HDMI module
- USB cables
- Case
- USB memory stick with software
- Polishing cloth
- User Guide
- Power adaptor
- HDMI to HDMI cable

Check first that the box has no damage to its exterior. In the event of damage, contact the delivery company and file a notification of damage.

Open the box and take out the MagniLink S; remove the white packaging. For further information see Installation.

4.1 Packaging

The packaging consists of the following packaging material (LVI recommends that the packaging be kept in case you have to return the system or you need to move it):

- Packing material (EPE)
- Box (corrugated cardboard)
- Polyethylene (LDPE4)

For sorting and collection of packaging material, please call the packaging collection customer service department for information on the rules that apply in your municipality.

5 Accessories



Larger external control box (MLV-A110A).



MagniLink Reading Table (MLRT), portable reading table.



LVI Monitor 19" (MLS-A103) and 23" (MLS-A104).



MagniLink Docking Station 2 (MLS-A106) with reading table.



MagniLink Docking Station 2 (MLS-A107) with a plate.



MagniLink Docking Station 2 (MLS-A108) with table clamp.




Text To Speech for PC (TTS-PC) or Mac (TTS-MAC).

6 Hardware

6.1 Installation of hardware



Open the camera's arm and pull out the camera's supporting legs. It is not possible to manipulate the camera's arm until the supporting legs have been extended.

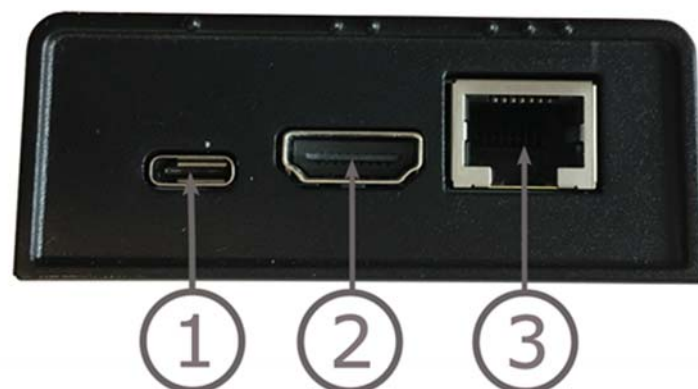

The USB/HDMI module has several connection options.

1. USB connection

In addition to transferring images to a computer, the USB connection is used to provide power to the system.

2. HDMI connection

Here the camera can be connected to a monitor that supports HDMI or DVI.


3. External control panel

An external control panel can be connected via this input. If the camera is linked to a docking station, the control panel must be connected before you attach the USB connection.

6.2 Installation of docking station


If you bought a MagniLink Docking Station, follow the instructions below to install it.

A MagniLink Docking Station consists of a camera holder and a reading table, plate or table clamp. If it is the variant with a table clamp, fasten it to the table.

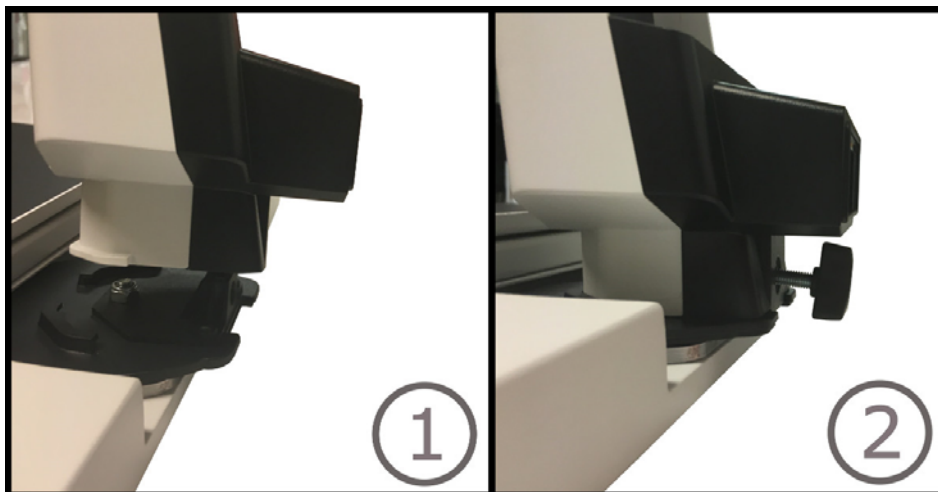

1 Place the holder in the reading table, plate or table clamp.


2 Fasten the holder with the screw.

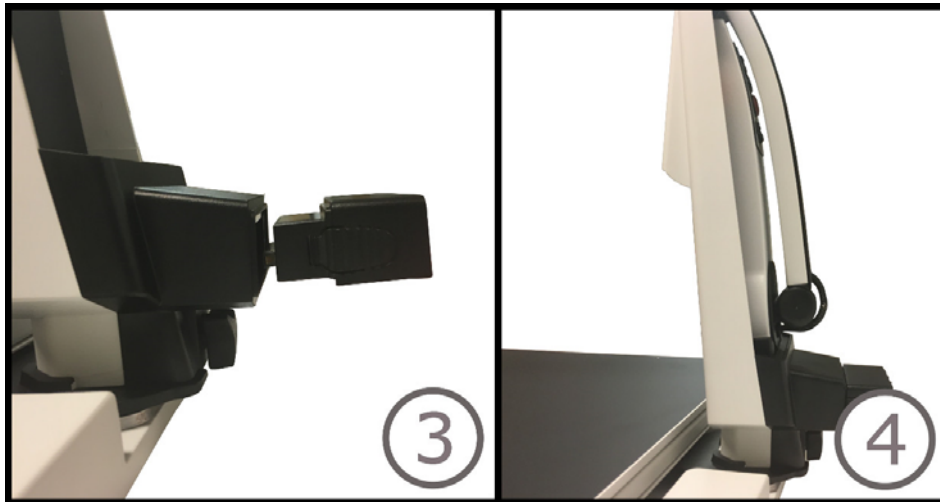

3. Remove the USB / HDMI module from the camera and insert it into the holder.
4. Place the camera in the holder.

6.3 Using the hardware

6.3.1 Control box

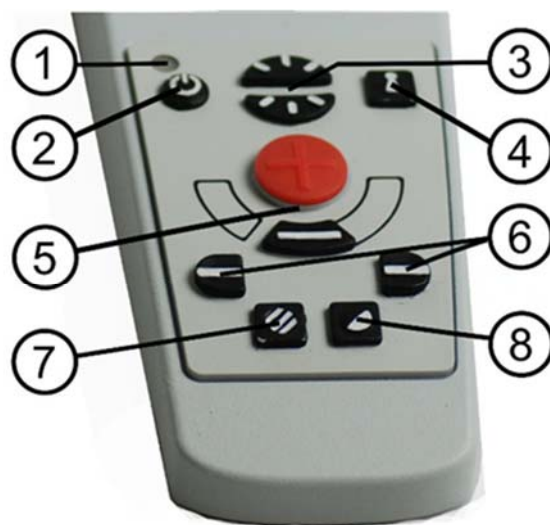

1. LED

A green light indicates that the system is on. A flashing green light means that the system is starting up. A slowly flashing yellow light indicates standby. A red light indicates that something is wrong in the system. See the chapter on Troubleshooting page 50.

2. Power on/off

Press to start or shut down the system.

3. **Lighting configuration**

Used to configure the light intensity in order to optimize reading conditions. Press the upper half to increase the light intensity and the lower half to decrease it. For systems that have LED lighting, the intensity is adjusted by holding down the button (7).

4. **Autofocus on/off**

Press the button once to lock the focus. This can be used when you wish to write by hand. Autofocus is not affected by the pen or your hand. Press the button a second time, to reactivate autofocus.

5. **Zoom control**

Used to set the zoom. To reduce the zoom, press the lower key (minus) and to increase the zoom, pressure the upper key (plus).

6. **Reference line/Bar**

Adjust the position of the reference line. Press the right-hand button to move the horizontal line down. Hold the button in to change the reference line to vertical and move it to the left. The reference line then becomes a horizontal bar which diminishes the viewing area. To make the bar vertical, press the button again. The left button gives the opposite result.

7. **Picture mode**

Press the button to switch the picture mode between natural colours and grey scale.

Lighting adjustment

Hold the button down to temporarily enable adjustment of the lighting via the lighting configuration (3). When the lighting configuration (3) is used to adjust the lighting, there is no switching between colour and grey scale (only possible if the system has LED lighting).

Mirroring

A mirror image can be obtained by holding the button down for three seconds.

8. **High contrast modes with artificial colours**

Press the key to switch between the various colour combinations.

Rotation

By holding the button down for three seconds, the image can be rotated 180 degrees.

6.3.2 Position sensor



MagniLink S is equipped with a position sensor that is used to switch between reading mode (the user points the camera down and flips the lens into place) and distance mode (the user opens the lens). The settings for both modes are automatically saved when you switch from one to the other.

Settings that are automatically saved for these modes are:

- Artificial colours
- Negative/Positive/Natural colours
- Light
- Magnification level
- Reference line/Bar
- Rotation of camera image
- Mirroring of camera image

7 MagniLink PCViewer

MagniLink PCViewer is a computer programme that displays images from a MagniLink camera system. The programme can control the camera properties, such as magnification level, artificial colours and brightness. If the system supports OCR, the OCR can process a text and read it aloud through speech synthesis (text to speech). The programme can also share the computer screen with ZoomText or SuperNova software.

7.1 Minimum system requirements

- Microsoft Windows 7.
- 2 integrated USB 2.0 ports.
- i3 processor, 1 GB RAM (32 bit) 2 GB RAM (64 bit).

7.2 Recommended system requirements

- Microsoft Windows 7.
- 1 integrated USB 3.0 port.
- i5 or i7 processor, 4 GB RAM, external graphics card with at least 512 MB inbuilt memory.

LVI has tested the software on several computers without any problems, but there is no guarantee that the system will function on all computers that comply with the specifications. LVI recommends testing the software on the computer where the system is to be used, in order to ensure compatibility. Better performing computers will facilitate better camera performance.

7.3 Installation of software

NOTE



Installation requires administrator privileges on the computer in question and is performed on the user account currently in use.

Insert the USB stick containing the software into one of the computer's USB ports. Start the installation by clicking on the "**Setup**" file, which can be found in the "**MagniLink PCViewer**" folder.

When the installation program starts, follow the instructions on the screen to complete installation of the software, or see below for a more detailed description.




Select "Next" in the dialogue box "Welcome to the InstallShield Wizard for MagniLink PCViewer".




Read the text in "License Agreement", approve this by selecting "I accept the terms of the license agreement" and click on "Next" to start the installation.



If you have a camera with OCR functionality, then mark the language(s) for which you have a license. If you do not have OCR functionality, then do not select any languages. Then click "**Next**" to continue with the installation.

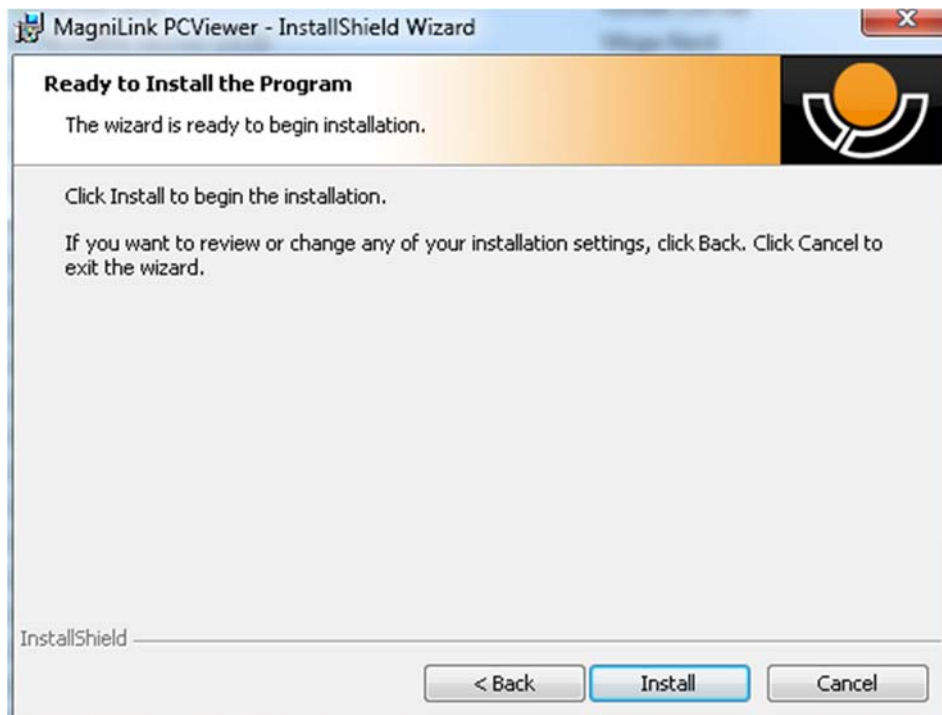

Select "**Install**" in the dialogue box "**Ready to Install the Program**".

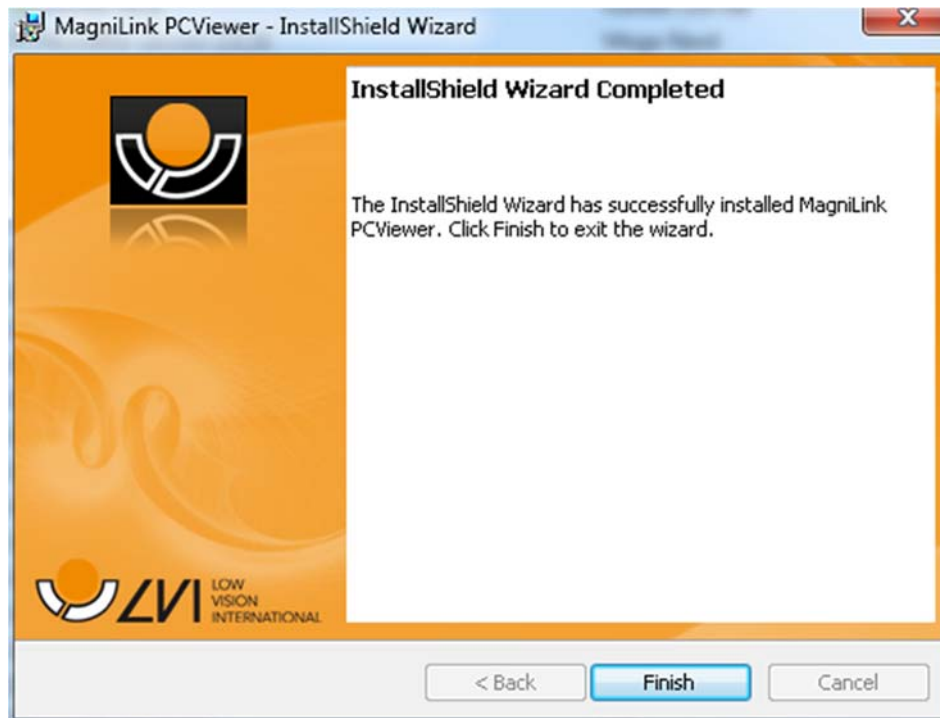

Select "Finish" in the dialogue box "InstallShield Wizard Complete".

7.4 Licensing


The camera system that is used with the software can be licensed with different functions.

- It is possible to license the system with or without OCR functionality.
- It is possible to license the system so that the software does not communicate with the internet.
- It is possible via the licence to remove the functions video recording and take picture.
- It is possible to license the system with voices in the following languages:
English (US), English (UK), German, French, Finnish, Italian, Dutch (BE), Dutch (NL), Swedish, Danish, Polish, Spanish, Portuguese, Norwegian, English (AU), Russian, French (CA), Arabic, Hebrew, Hungarian, Rumanian, Bulgarian, Czech, Slovakian, Croatian, Turkish, Greek.


7.4.1 Computer license

By licensing the software to a computer, you can use the software without a connected camera. To license the software you will need an activation key. Activation keys can be purchased from your LVI distributor.

To activate a computer license, launch the application without a camera connected. The following dialogue will be displayed.

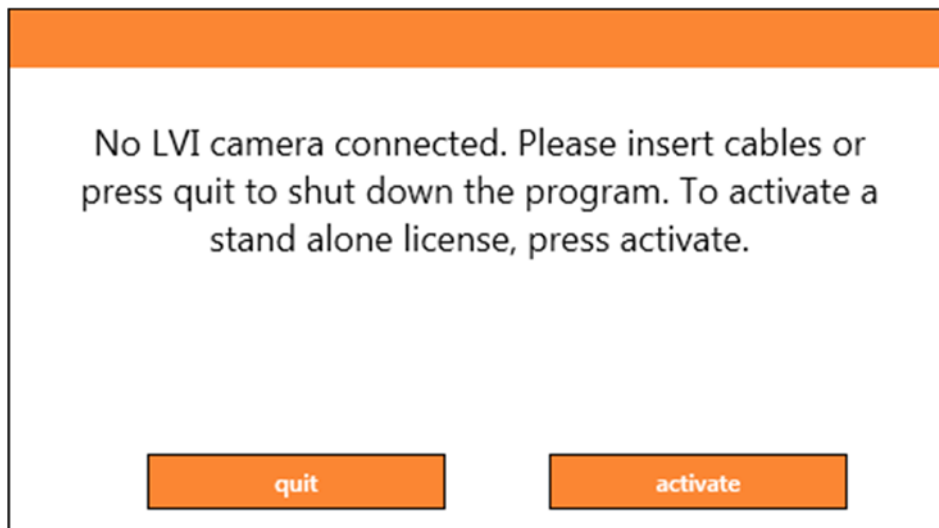

Click on "activate". If the application establishes contact with the activation server, the following dialogue box will be displayed.

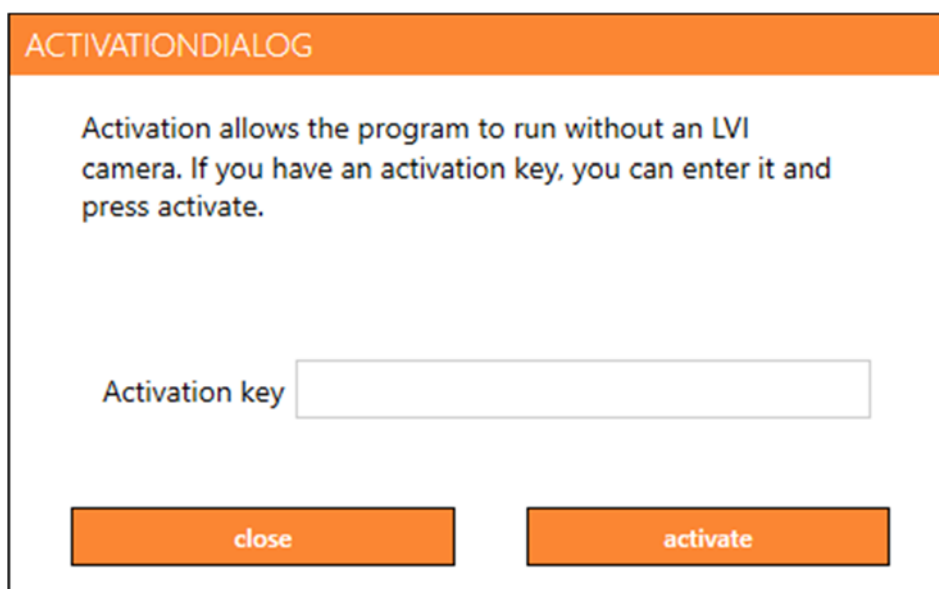

Enter your e-mail address and your activation key, then click on "activate". You can now start using the software without a camera.

ACTIVATIONNOINTERNETDIALOG

The application was unable to connect to the activation server. To activate on a computer without internet connection please follow the steps below.

Contact your LVI dealer and provide them with the following information:

- A. Your activation key.
- B. Your computer ID shown below.

You will then receive a new activation key that will work on your computer.

Computer Id ECD0A0E0DFEFBEFB

Activation key

close

activate

The following dialogue box is displayed if there is no internet connection or the application is unable to connect to the activation server. Follow the instructions and contact your LVI distributor in order to activate the applications.

7.5 Using the software

The software can be started from the icon on the desktop, from the start menu or using the short cut command Ctrl + Alt + S.

7.5.1 Functions via menus

The application consists of a window with tabs and groups of buttons. There is a tab for application settings and another tab for handling the camera. Depending on whether the system is licensed for OCR, there is also a tab for handling the OCR processed text.

7.5.1.1 Camera tab

The following tab will only appear if there are one or more cameras connected.



Change colours

Used to change between natural and artificial colours.



Decrease magnification



Increase magnification



Overview mode

Reduces magnification to overview mode. Click again to restore the magnification. When overview mode is active, a sight is displayed in the middle of the video window.

**Lock autofocus**



Locks the autofocus mode. This can be used when you wish to write by hand. In this way, autofocus is not affected by the pen or your hand. Press the button a second time to reactivate autofocus.

**Focus up**


Reduce focus distance. When this function is used, autofocus is turned off.

**Focus down**

Increase focus distance. When this function is used, autofocus is turned off

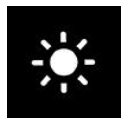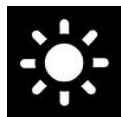**Reduce brightness/contrast****Increase brightness/contrast****Reference line/bar left/up**

Adjust the position of the reference line. Press the button to move the reference line to the left. Hold the button in to change the reference line to horizontal and move it upwards. The reference line then becomes a vertical bar which diminishes the viewing area. To make the bar horizontal, press the button again.

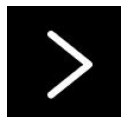**Reference line/bar right/down**

Same as the previous command but in the opposite direction.

**Reduce lamp brightness****Increase lamp brightness**



Take snapshot

A dialogue box "Save pictures" opens. A file name (with the day's date and time) and a folder (My Pictures) are automatically suggested. Just press "Enter" to save the picture. If you wish to save the picture somewhere else, or with another name or in another format, (the standard is JPG), you can change this manually. To save a picture quickly, hold down "Shift" when you press the button.



Show pictures

Press to view the saved pictures. Mark the picture you wish to view and press Enter or select Open. The picture is normally opened in Windows' standard picture-viewing programme where, among other things, the picture can be enlarged. To view the most recently saved picture quickly, hold down "Shift" when you press the button.



Start/stop video recording

Press to open the "Save video clip" dialogue box. A file name (with the day's date and time) and a folder (My video clips) are automatically suggested. Press Enter to save with the suggested name. If you wish to save the video in a different folder, or with another name, you can change this in the dialogue box. Stop the recording by pressing the button again. To start video recording quickly, hold down "Shift" when you press the button.




Show video recording

Press to view the saved video files. Mark the video you wish to view and press Enter or select Open. The video is normally opened in Windows' standard media player software. To view the most recently saved video recording quickly, hold down "Shift" when you press the button.



Rotate the camera image

Rotates the camera image clockwise. To rotate anticlockwise, hold down Shift when you press the button. To rotate steplessly, hold down Ctrl.

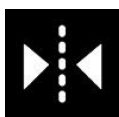

Mirror the camera image

Press again to reverse the mirror image.

**Freeze the camera image**

Press again to release the freeze.

**OCR process** (Only shown if the system has a licence for OCR)

When OCR processing is complete, the programme changes to the OCR tab and the text is read. If there is already text under the OCR tab this is replaced. Hold down Shift when you press the button to insert the text on a new page.

**Whole image** (Only shown if the system has a licence for OCR)

When this is active, the whole camera image is shown. Otherwise edit in the image, depending on how large the camera window is.

**Notepad**

Opens images from the camera in the notes tool.

**Next split screen mode**

Switches to the next split screen mode. In order to switch to the previous mode, hold down Shift when clicking on the button.

**Always on top**

Activates/deactivates "always on top"

**Help**

Opens the user guide in the standard application for showing PDF files.

7.5.1.2 OCR tab

The following tab is only shown if the system has an OCR licence.



Save documents

A dialogue box opens. A file name (today's date and time) is suggested automatically. It is possible to save the text as a Word file (.docx), PDF file, text file (.txt), WAV file (.wav) or zip file.




Open document (with OCR processing)

A dialogue box opens. From the dialogue box, select the file you wish to open. It is possible to open PDF files and image files, .bmp, .jpg, .png, .gif and .tiff, as well as saved zip files.




Change colours

Changes colours of the OCR processed text.

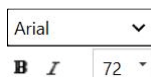


Font

Used to change the font of the OCR text. It is also possible to choose bold and italic text. The used font does not impact the OCR result.

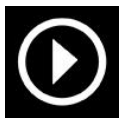

Start/Pause reading

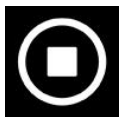


Stop reading

The next reading will begin from the start of the text.




Change display mode

Changes the display mode for OCR processed text. The display modes available are "page view", "row view", "word view", "image view", "image and word" and "image and row".

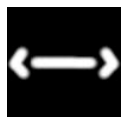

Silent display mode

Switches to silent display mode. There are two silent modes; one where the text is moved horizontally on one line and one where the text is moved vertically in a column.

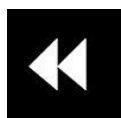

Read previous

With this button, the user can go backwards through the text. If the reading mode is set as character, one character is read for each keystroke. If the reading mode is word, you move backwards through the text word by word. If the reading mode is sentence, you move backwards one sentence. If the reading mode is block, you move backwards one block.



Read next

With this command, the user can move forward in the text. If the reading mode is set as character, one character is read for each keystroke. If the reading mode is word, you move forward through the text word by word. If the reading mode is sentence, you move forwards one sentence. If the reading mode is block, you move forwards one block.



Switch reading mode

This command selects the reading mode that is to be employed when the "Read previous" or "Read next" command is entered. The modes available are: character, word, sentence, block and view. The modes that can be selected change depending on the display mode.



Volume and speed

The upper slider sets the volume and lower regulates the speed of the speech.



Pages

Shows all OCR processed pages. Click one of the pages to change page.



Notepad

Opens the OCR page in the Notes tool. Hold shift to open all pages.



Next split screen mode

Switches to the next split screen mode. In order to switch to the previous mode, hold down Shift when clicking on the button.



Always on top


Activates/deactivates "always on top"



Help

Opens the user guide in the standard application for showing PDF files.

7.5.1.3 Notepad tab

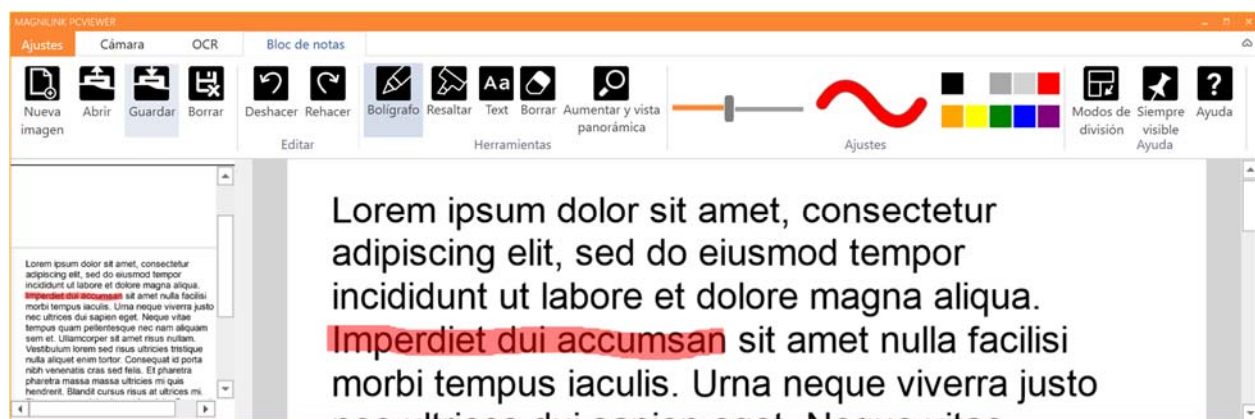

The notes tool consists of three components.

- At the top is a menu bar with buttons.
- On the left is a list of pictures. By clicking a picture, you can navigate to it. You can delete an image by clicking in the upper right corner.
- A surface where you can draw, enlarge and pan in the picture.



New image

Opens a new blank page.



Open

Opens the bild. The system can open images in .jpg and .png format.

**Save image**

The system can save images in .jpg and .png format.

**Delete**

Deletes the current page.

**Undo****Restore**

Restores after pressing undo.

**Pen**

Draw with a pen.

**Highlight**

Highlight with a highlighter.

**Text**

Insert text. You can move text around after inserting.

**Delete****Magnify and pan**

Magnify and pan the image.

**Size**

Sets the size of pens and text.



Color

Changes the pen and text colour



Next split screen mode

Switches to the next split screen mode. In order to switch to the previous mode, hold down Shift when clicking on the button.



Always on top

Activates/deactivates "always on top"



Help

Opens the user guide in the standard application for showing PDF files.

7.5.2 Functions via the keyboard

Most functions can be handled using the keyboard. The shortcut commands can be changed in settings. Below are the standard settings for shortcut commands.

F1	Natural colours
F2	Positive colours forwards
Shift + F2	Positive colours backwards
F3	Negative colours forwards
Shift + F3	Negative colours backwards
Up arrow	Increase magnification
Down arrow	Reduce magnification
Ctrl + O	Overview mode
Ctrl + J	Autofocus on/off
Shift + up arrow	Reduce focus distance
Shift + down arrow	Increase focus distance
Right arrow	Increase brightness/contrast
Left arrow	Reduce brightness/contrast

Ctrl + left arrow	Reference line left/up
Ctrl + right arrow	Reference line right/down
Ctrl + 6	Reduce lamp brightness
Ctrl + 7	Increase lamp brightness
Ctrl + R	Rotate image clockwise
Ctrl + Shift + R	Rotate image anti clockwise
R	Rotate image clockwise steplessly
Shift + R	Rotate image anticlockwise steplessly
Ctrl + I	Mirror camera image
Ctrl + F	Freeze camera image
Ctrl + A	Always on top
Ctrl + H	Hide menus
F12	Open user guide
F9	Settings
F11	Standard settings
F7	Take a picture and open in Notepad
Ctrl + C	Copy image or text
Ctrl + M	Minimise application
F8	Next view
Ctrl + F8	Previous view
Ctrl + Minus	Reduce menu size
Ctrl + Plus	Increase menu size
F5	Save picture
Shift + F5	Save picture quickly
F6	Open picture
Shift + F6	Open picture quickly
Ctrl + F5	Start video recording
Ctrl + Shift + F5	Start video recording quickly
Ctrl + F6	Open video
Ctrl + Shift + F6	Open video quickly

The following commands are only available together with cameras that are licensed for OCR.

F4	Start OCR The text from OCR processing replaces existing text.
Shift + F4	Start OCR enter The text from OCR processing is entered on a new page.
Ctrl + L	Whole image Changes between whole image and cropped image.
Ctrl + V	Paste The pasted text replaces existing text.
Ctrl + Shift + V	Paste and add The pasted text is added to a new page.
Ctrl + S	Column selector Activates or deactivates the column selector. When the column selector is active, only those columns that are intersected by the column selector are OCR-processed.
V	Changes between camera tab and OCR tab
Alt + F5	Save documents
Alt + F6	Open documents
Up arrow	Increase magnification
Down arrow	Decrease magnification
Shift + Plus	Increase font size
Shift + Minus	Decrease font size
1	Start reading
2	Stop reading
B	Change display mode
N	Silent mode
3	Read previous
4	Read next
5	Switch reading mode
Page Down	Next page If several pages have been scanned, you move to the next page.

Page Up	Previous page If several pages have been scanned, you move to the previous page.
Delete	Remove current page If several pages have been scanned, you remove the current page.
8	Reduce volume
9	Increase volume
6	Reduce speaking speed
7	Increase speaking speed
Ctrl + 0	Next voice
A	Pan left (If the document is magnified)
D	Pan right (If the document is magnified)
W	Pan up (If the document is magnified)
S	Pan down (If the document is magnified)

The following commands are used to control Notepad functions.

Ctrl + N	New Page
Ctrl + O	Open image
Ctrl + S	Save image
Ctrl + Z	Undo
Ctrl + Shift + Z	Redo
Ctrl + P	Select pen tool
Ctrl + L	Select the highlighter tool
Ctrl + T	Select text tool
Ctrl + E	Select the eraser tool
Ctrl + F	Select the pan tool

7.5.3 Functions via the mouse

Some functions can be controlled by the mouse.

In camera mode.

- The scroll wheel can be used to zoom in or out.

In OCR-mode


- Left mouse button is used to start or pause the speech.
- By rolling the scroll wheel, the picture and font size can be increased or reduced.
- Press and hold the right mouse button or the scroll wheel and move the mouse to navigate the picture (only possible if picture has been magnified).

It is possible to open a document by dragging it into the application.

7.5.4 Information and settings window

The Information and settings window consists of a number of different tabs that contain application information and alternative settings.

7.5.4.1 About the application

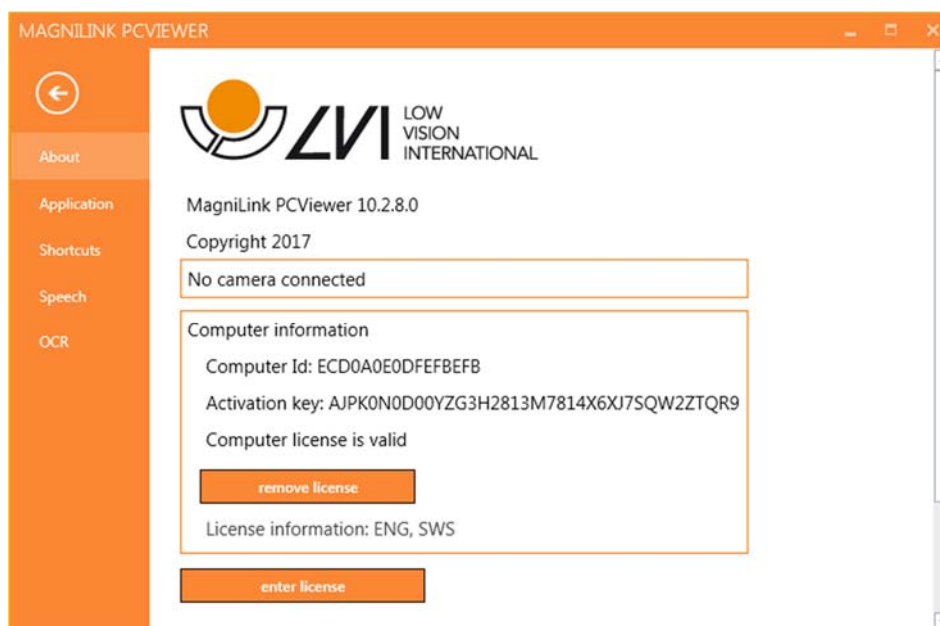


Information regarding the software version and license can be found under this tab.

To deactivate a computer license, press "remove license". The license will then be deactivated on the server and it will be possible to activate it on another computer.

If you want to update the camera license or activate a computer license, you can do it by pressing the "enter license" button.

7.5.4.2 Application Settings

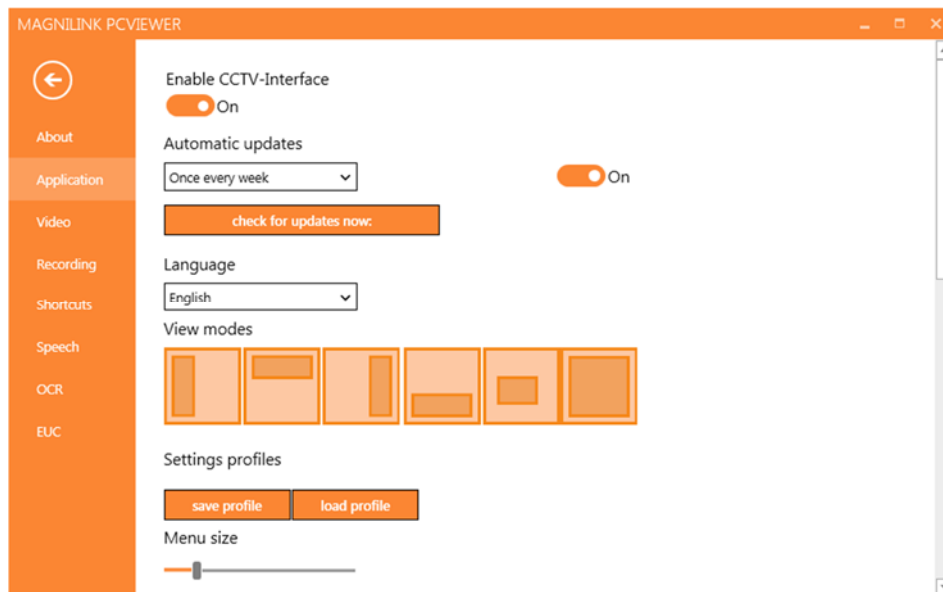

Activate CCTV interface

If this function is activated, MagniLink PCViewer supports working with the magnification software from ZoomText and SuperNova.

Automatic updates

This can be set if MagniLink PCViewer is to search for updates. You can also set how often it will search: "Once per day", "Once per week" or "Once per month". You can also choose to make updates manually by clicking "search for updates now" (note: an internet connection is needed for both automatic and manual updates).

Language

Change the language of the software. The software is set to use the same language as the computer's default. If this language is not available, English is automatically selected as default.

Available languages:

- English
- German

- Danish
- Spanish
- Finnish
- French
- Dutch
- Norwegian
- Swedish

Split screen mode


It is possible to choose the mode you want when changing split screen mode.

Settings profiles

It is possible to save and load settings profiles.

Menu size


The size of the menus and buttons can be changed here.

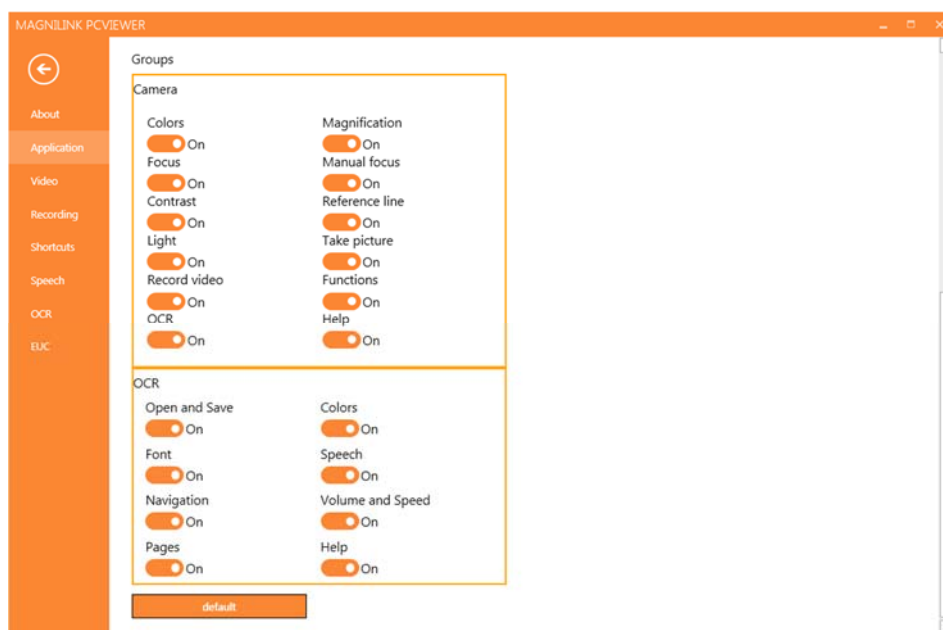

Groups

It is possible to choose which groups of buttons appear in the menus.

7.5.4.3 Video Settings

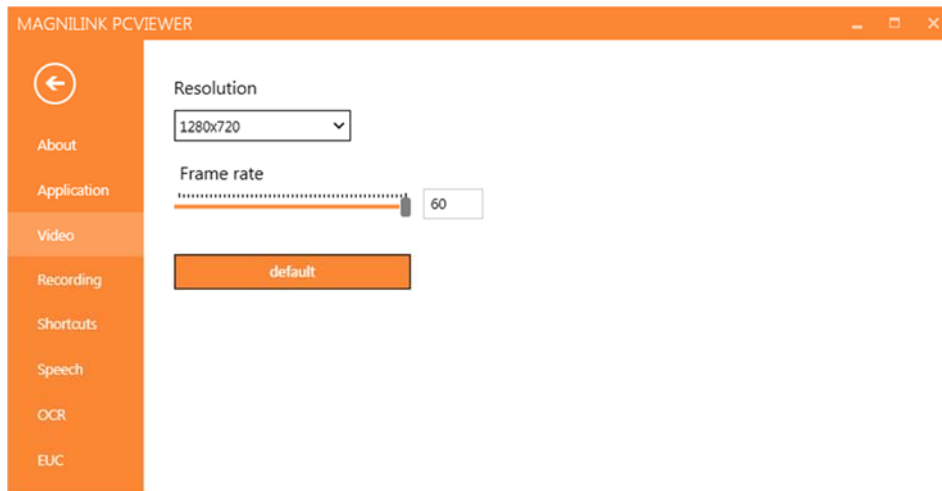


Resolution

Select from the camera's various resolution options. At resolution 1280x720, the frame rate can be set up to 60 Hz. At resolution 1920x1080, only 30 Hz is possible.

Frame rate

This setting allows the user to limit the frame rate. For the best video quality, the frame rate should be 60 Hz, but it may be appropriate to set the frame rate at 30 Hz for computers with a lower level of performance.

7.5.4.4 Recording Settings

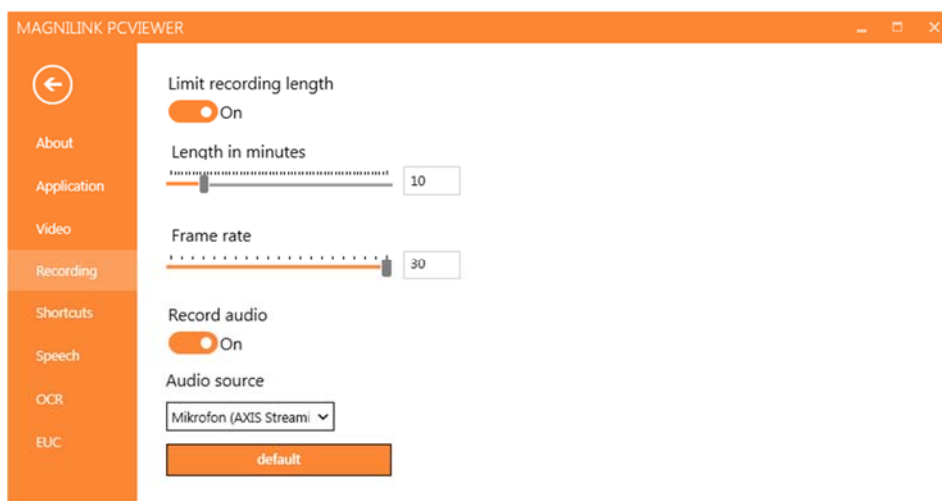

Limit recording length

Activate this function if you wish to limit the length of recording.

Length in minutes

If you chose to limit the recording length, you can set the limit in minutes here, from 1 to 60 minutes.

Frame rate

The recording frame rate, the number of images recorded per second, can be set from 10 to 30 frames per second. A lower frame rate means a poorer quality video recording, but takes up less space on the hard disc.


Record sound

If your computer has a microphone (internal or external), sound can be added to the recording.

Sound source

Select which microphone is to be used to record sound for the video. Normally only one sound source may be selected.


7.5.4.5 Short cut commands

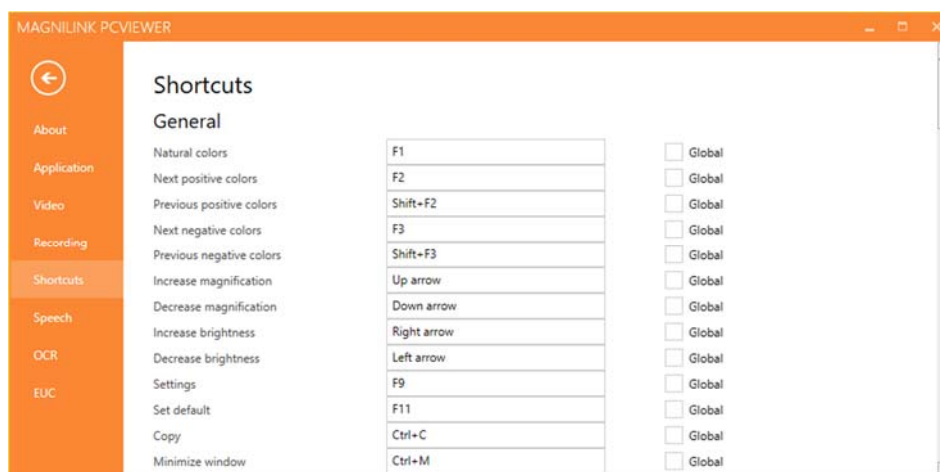

Here the short cut commands used for the application can be changed. To change a short cut command, tab to the command that is to be changed and enter the new keyboard combination. It is possible to set two different combinations for the same command. If a combination already being used for another command is entered, the other command will be erased. A new keyboard combination will need to be entered for the other command.

It is also possible to make a keyboard shortcut global. A global keyboard shortcut can be used also when the application is inactive.

7.5.4.6 Speech Settings

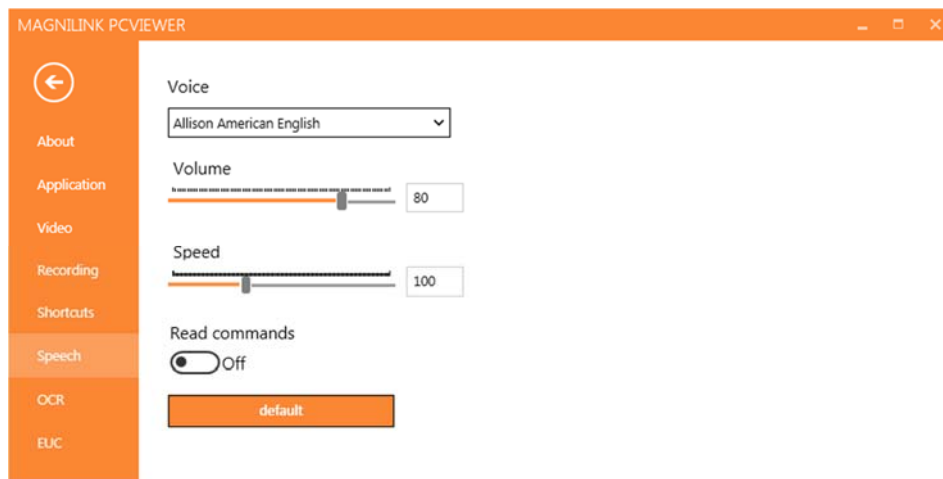

Voice

If several voices are available, the user can choose between them.

Volume

Setting the sound level of the voice.


Speed

Setting the speaking speed.

Read command

When this is checked, the application reads out the menus when you use them.

7.5.4.7 OCR Settings

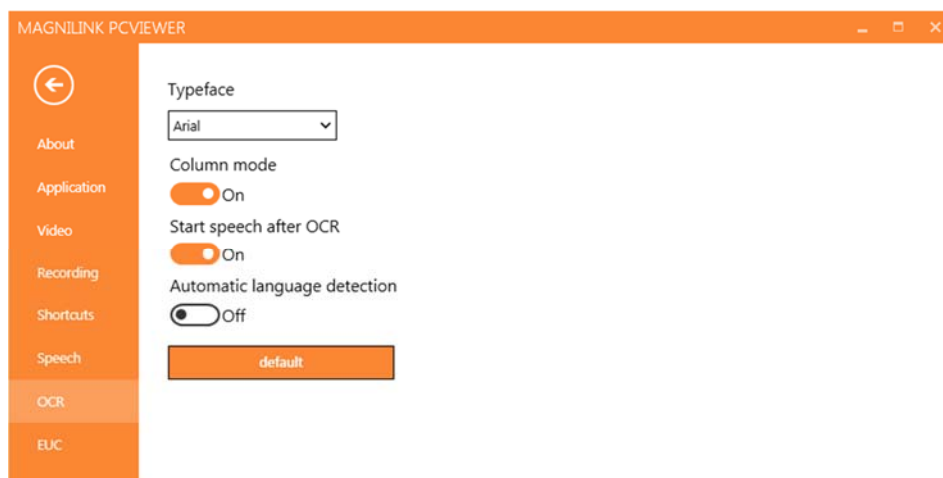

Font

Here the user can set which font is to be used to display the OCR

processed text. The font that is used has no effect on the result of the OCR processing.

Column mode

When this setting is active, the OCR engine processes the text by column, which is useful if text from a newspaper is to be OCR processed.


Start speech after OCR

If this box is ticked, text will automatically be read aloud after an image has been OCR processed.

Automatic language identification

When this function is activated, the application attempts to identify the language that has been OCR processed and change the voice accordingly.

7.5.4.8 EUC

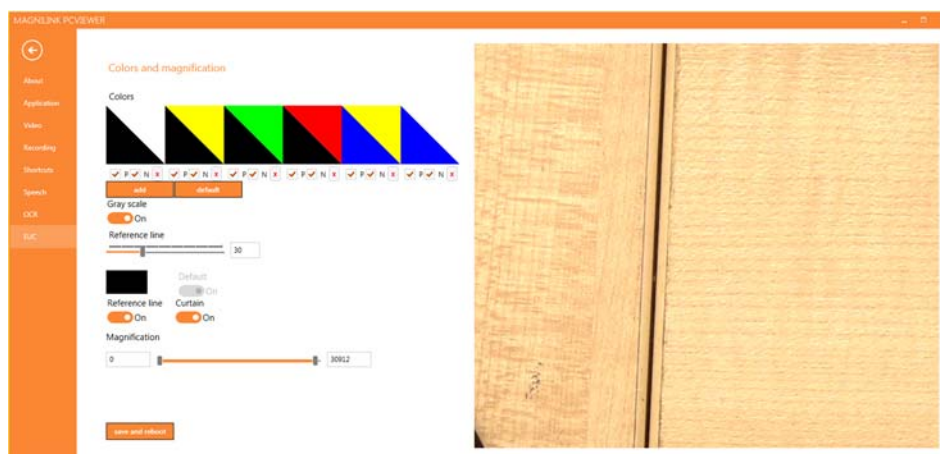

The camera can be configured under this tab. If you wish to save changes in the camera, press "Save". When you do this, the camera restarts and the configuration has been saved.

Artificial colours

It is possible remove and change the cameras artificial colours.

- To remove a colour combination, press X
- To add to a colour combination, press enter.
- To change a colour combination, press one of the triangular zones and a colour dialogue will appear. Select colours and press OK.
- To remove either the positive or the negative component. Click out box P or N.

- To restore the colours, press Standard.

Grey scale

Choose if you wish to have grey scale or not.

Reference line

Set the size and colour of the reference line and choose if you wish to have reference line, bar or both.

Magnification

Set the maximum and minimum magnification.

7.5.5 Quality of OCR processing

In order to optimize the results of the OCR conversion, you should bear in mind the following.


- Ensure the magnification is sufficient. Very small text produces poor results in the OCR software. A minimum of 12pt is recommended for the lowest magnification on the reading camera.
- Avoid glossy paper as this results in reflections in the image. This impairs the results of the OCR conversion. If you intend to use glossy paper, we recommend that the light on the unit is switched off.
- Certain fonts can be difficult for the OCR engine to interpret.
- It is important that the text contrast is sufficient.
- It is important that the OCR engine's language is the same as that of the text.
- The camera's resolution should be set to at least 1280x720, see Video settings for more information.

7.6 Update software


When a new version of the software is released, an update can be made. Follow the instructions below when the application finds a new version.




Click **"Next"** in the dialogue box **"Welcome"**. If there is an update, the following dialogue box is shown.

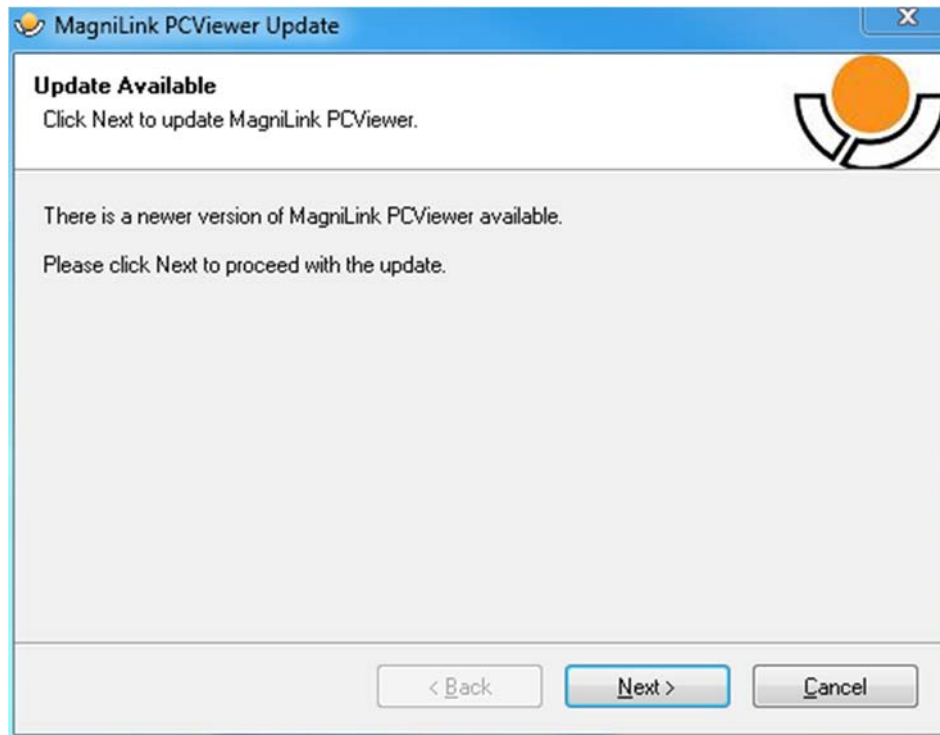


Click **"Next"** in the dialogue box **"Update available"** to start downloading the update.

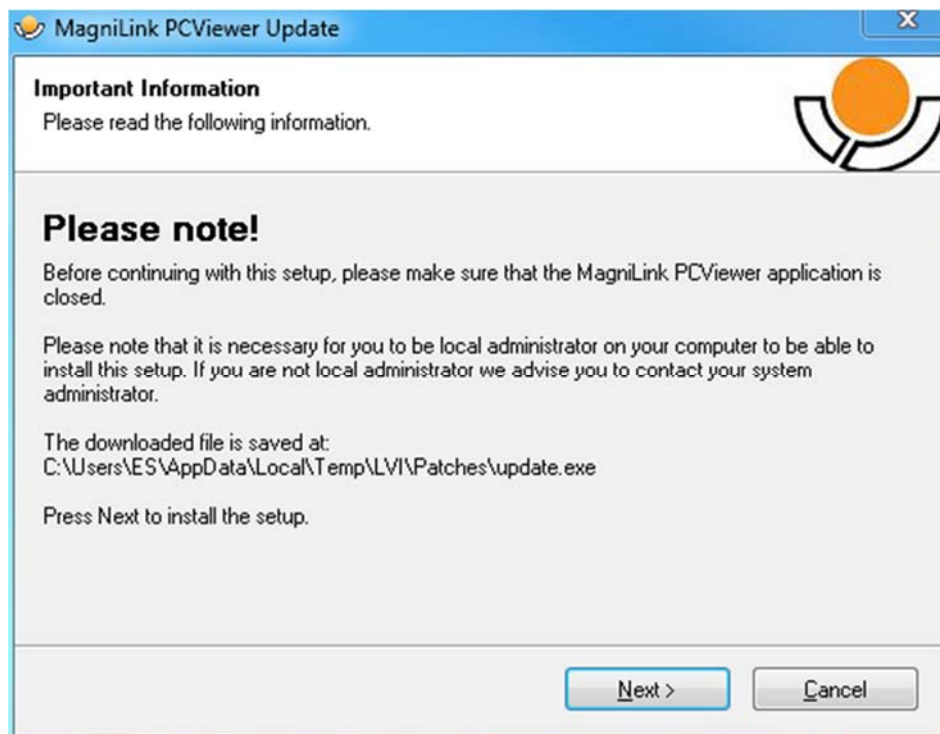

Ensure that you close MagniLink PCViewer before clicking **"Next"** in the dialogue box **"Important information"**.

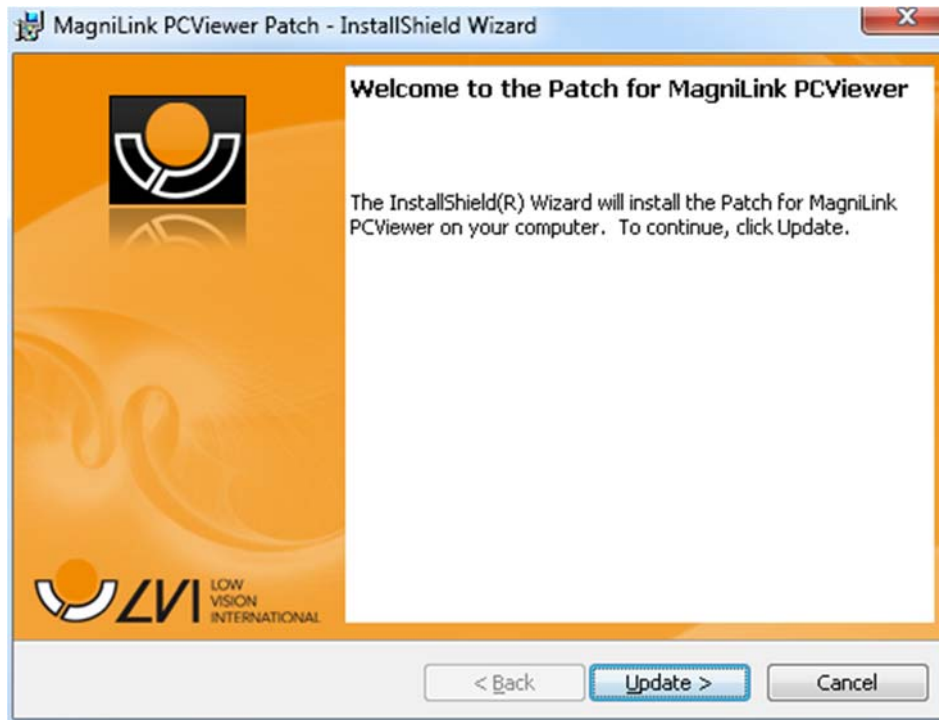

Click "Next" in the dialogue box "Welcome to the Patch for MagniLink PCViewer".

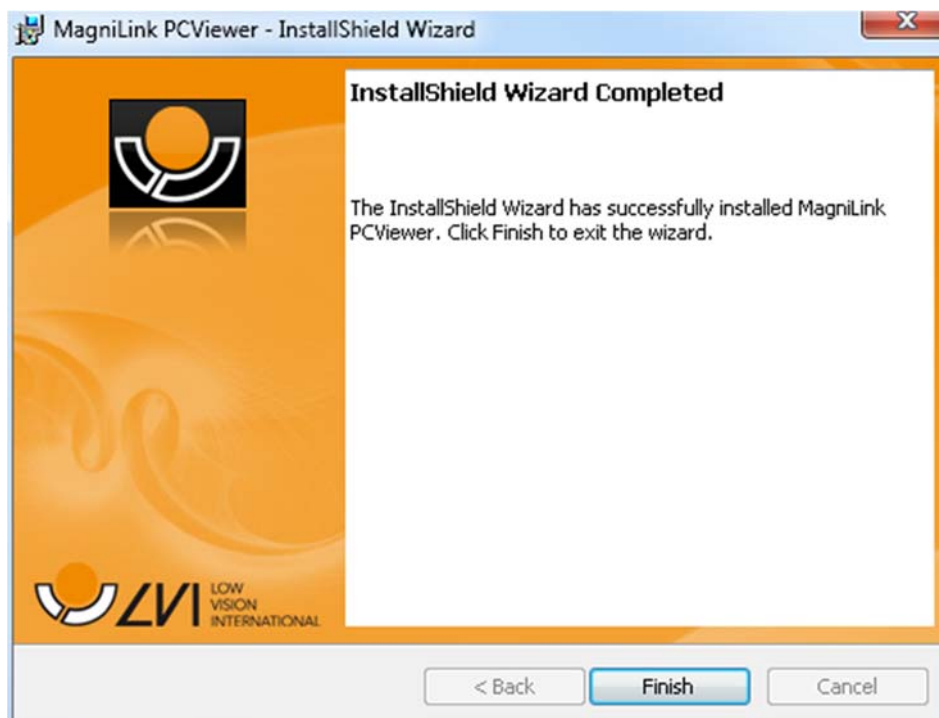

Click "Finish" in the dialogue box "Installshield Wizard Completed".

8 Working with the reading camera

Your reading camera is now ready to use. It is important that the workplace around your MagniLink is comfortable, and that you sit correctly. This is so that you can use the camera for as long as possible without putting undue strain and stress on your neck and back.

You should bear the following in mind:

- Your chair should be positioned close to the table on which the equipment is located, and you should not sit in the chair for too long. We recommend a good office chair with adjustable height.
- The seat of the chair should be at a height that does not cause your legs to become numb; legs should be at an angle of 90 degrees and you should rest your feet on the floor.
- The reading table should be at a height at which you can rest your lower arm horizontally on it. It is important that the table is neither too high nor too low for you. The ideal solution is if the screen is slightly below eye level and in as straight a line in front of you as possible.
- Shut out daylight. Lights in the ceiling, table lamps and light from windows can cause reflections on the monitor. For this reason, avoid placing lights where they shine directly at the screen or at yourself.

9 Troubleshooting

If the indicator light is flashing:

A red light or a flashing red light indicates a system error. Restart the system by disconnecting and reconnecting all cables and rebooting the system. If a red light is still displayed, contact your service technician.

10 Technical information

Magnifying area HD camera	1,4 - 75 x (15,4" 16:9 screen)
Magnifying area FHD camera	1,1 - 75 x (15,4" 16:9 screen)
Power output	5 W
Power output, standby	1.7 W
Weight	1.9 kg
Refresh rate	60 Hz
Dimensions BxDxH	350x305x390 mm
Operational temperature	0 to 30°C
Storage and transportation temperature	-20 to 60°C



Disposal of old electrical and electronic equipment (applies for EU countries and all other European countries that have specific collection systems)

If the product or its packaging has this symbol on it, it is not to be handled as regular domestic waste. It should instead be deposited at a suitable collection centre for the recycling of electrical and electronic equipment. By ensuring that this product is properly taken care of, you are helping to prevent the negative impact on health and the environment that might otherwise result from inappropriate disposal of this product. Recycling helps in the management of our natural resources. For more detailed information regarding the recycling of this product, you can contact your municipality, your waste management company or the store where you purchased the product.

11 Declaration of Conformity

DECLARATION OF CONFORMITY

Manufacturers name: *LVI Low Vision International AB*

Manufacturers Address: *Verkstadsgratan 5
S-352 46 Växjö
Sweden*

Web: *<http://www.lviglobal.se>*

SRN (Single Registration Number): *SE-MF-000004178*

Basic UDI-DI (GMN): *7330724MLSEC*

Brand name or trademark: *MagniLink*

Product code: ***MLS-HD-2, MLS-FHD-2***

Classification: *Class 1*

Type of equipment: *Reading Aid*

Conformity assessment route: *LVI Low Vision International AB uses the following
procedures for the CE-labeling of their products
according to the Regulation MDR 2017/745:
Class [1]: EC conformity declaration according to
Annex I + Annex II + Annex VII*

This declaration of conformity is issued under the sole responsibility of LVI Low Vision International AB. We hereby declare that the medical device(s) specified above meet the provision of the Regulation (EU) MDR 2017/745 for medical devices. This declaration is supported by the Quality System approval to ISO 9001 issued by Intertek. All supporting documentation is retained at the premises of the manufacturer.


☒ Manufacturer

☐ Manufacturer's authorized representative

Date

20-05-2021

Signature

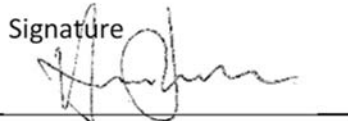

Position

Managing Director

Clarification

Henrik Blomdahl

Alphabetical index

About LVI	5
About the application	38
Accessories	12
Application Settings.....	39
Camera tab	26
Cleaning	8
Computer license	24
Control box.....	17
Declaration of Conformity	52
EUC.....	44
Functions via menus.....	26
Functions via the keyboard.....	34
Functions via the mouse	38
Hardware.....	15
Information.....	8
Information and settings window.....	38
Installation of docking station	16
Installation of hardware.....	15
Installation of software	20
Introduction to the MagniLink S.....	6
Licensing	23
MagniLink PCViewer	20
Minimum system requirements.....	20
Notepad tab	32
OCR Settings	43
OCR tab	30
Packaging.....	11
Position sensor	19
Quality of OCR processing	45
Recommended system requirements.....	20
Recording Settings.....	41
Reuse, installation and maintenance	8
Safety information	8
Short cut commands	42
Speech Settings.....	43
Technical information.....	51
Troubleshooting.....	50
Unpacking	10

Update software	46
Using the hardware	17
Using the software.....	26
Video Settings	41
Working with the reading camera.....	49